



# Moving Front Integrator Class 5



## SAP / Monitoring



### Simple and easy to interpret .....


REF: 105.003.1000 Moving Front Integrators 134°C-3,5 min Unit: 1000 pieces

Integrators can be used in addition to the batch control Helix which allows you to release a load. The moving front integrators Class 5 gives detailed information about the sterilization cycle.

The Sterintech™ Moving Front Integrators are working with a medium which becomes a fluid the moment that sterilization conditions have been met. The fluid is absorbed by the filtration paper inside of the Integrator over time. After 66% of the 3,5 minutes holding time have been reached at 134°C the blue fluid is reaching the acceptance window. An advantage is that you don't have to interpret the colour change of it. Just see where the blue fluid has reached i.e. Reject or Accept. It can be as simple as that.

#### Characteristics:

- Shelf life: 2 years after production
- Packed in white aluminium re-closable bags
- Conform to ISO 11140-1 : Class 5
- Very easy interpretation of result
- Complete indicator area sealed off by foil
- Non-Toxic / Lead Free

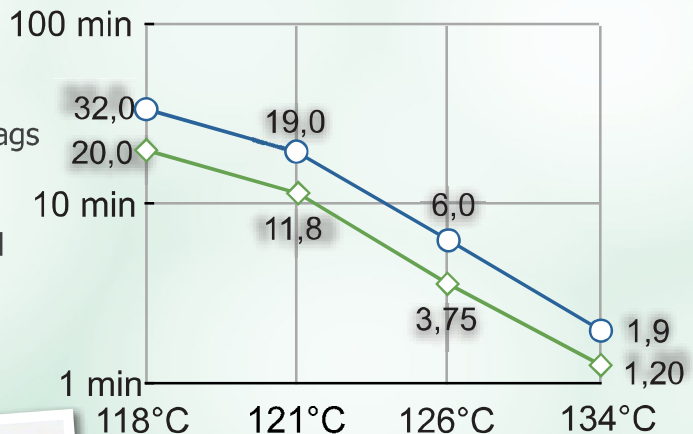

○ MFI Class 5 Reaching "Pass"  
◇ GeoBac. Stearoth. Killing Time